

ACS COMFORT EYE DROPS**COMPOSITION:**

Carmellose Sodium BP..... 0,5 % w/v
 (Carboxymethyl cellulose sodium)
 Polyhexamethylene Biguanide..... 0,0001 % w/v (as preservative)
 Sterile aqueous baseq.s

04/03/2025

Carmellose sodium eye drops soothes and gives comfort to the eyes.

DIRECTIONS FOR USE:

- Instill one or two drops into each eye when required, or use as directed by your eye-care practitioner.
- Blink several times.
- Replace the cap tightly after use.

CONTRA INDICATIONS:

- Do not use if you are allergic to any of the ingredients.
- If you experience any eye discomfort, irritation, pain, redness or blurred vision consult your eye-care practitioner immediately.

PRECAUTIONS:

- Do not use if tamper evident ring is broken before first use.
- Store between 5 °C to 30 °C.
- Do not touch the nozzle tip to any surface since this may contaminate the solution.
- Do not use after the expiry date.
- Use the solution within 90 days after opening the bottle.
- Discard the anti-tamper ring from the bottle to avoid falling off when applying drops.

**FOR EXTERNAL USE ONLY****WASTE DISPOSAL:**

- Pour unused Eye drops down a drain.
- Dispose of empty bottle in accordance with local waste disposal regulations.

License Holder:

ACS Pharma Access Health Solutions (Pty) Ltd
 Cnr Barbara Roand & North Reef Road,
 Elandsfontein, Germiston, 1429

**Manufactured by:**

Stericon Pharma Pvt. Ltd, Plot No. 9R, Sub layout of Plot No 9,
 1st Phase, Bommasandra industrial Area, Bangalore - 560 099,
 Karnataka, India



AN EN ISO 13485 CERTIFIED COMPANY



MYM STC, S.L. Avda. de los Rosales 32, 28935 Móstoles Madrid, Spain
 Email: direccion@mym-stc.com