




# ACS

## SALINE SOLUTION PLUS






**ACS SALINE SOLUTION PLUS** is a sterile, pH balanced, aqueous solution used as a rinsing solution for **soft, hard and scleral lenses**.

### Benefits Include



-  Minimises Irritation: Effectively removes residual disinfectant from lenses, reducing eye irritation.
-  Versatile Use: Compatible with all lens types and dissolves protein removal tablets.
-  Enhances Safety: Maintains lens hygiene and reduces the risk of eye infections.

### Usage Instructions

-  3 drops of **ACS SALINE SOLUTION** should be applied to each contact lens after disinfection<sup>1</sup>
-  The contact lenses should always be **handled with clean, dry hands**<sup>1</sup>
-  Solution should be used within **30 days after opening** the bottle<sup>1</sup>
-  Solution should **not be reused. Don't use after expiry date.**<sup>1</sup>
-  The solution should **not be injected or taken orally**<sup>1</sup>

- Available without prescription
- Suitable for contact lens wearers
- Claimable from selected medical aids